

## SIENET™ LiteBox™ Workstation KEYBOARD COMMAND REFERENCE

### KEY


### FUNCTION

- ⌘ (command) ..... Combined with another key, performs "shortcuts."
- Allows you to window/level an image regardless of which tool is selected.
- When used with the Magic Glass tool, allows you to window/level the contents of the Magic Glass, except for Equalize.
- CONTROL ..... Resizes the Magic Glass.
- Resizes the Window/Level and Identify subsection.
- END ..... Scrolls to the end of an exam.
- HOME ..... Scrolls to the beginning of an exam.

### KEY

### FUNCTION

- OPTION ..... Pans an image when using the Zoom tool.
- Zooms an image when using the Pan tool.
- Changes an image's window identify range when using the Window/Level tool.
- Allows you to temporarily use the Pixel Lens tool when using the Pointer tool.
- Zooms the contents of the Magic Glass when using the Magic Glass tool.
- Temporarily changes to Pointer tool.
- PAGE DOWN ..... Scrolls to the next bank of images.
- PAGE UP ..... Scrolls to a previous bank of images.
- POWER ..... Starts the Macintosh.
- SHIFT ..... Controls propagation of image changes when using Window/Level, Zoom, Pan, and Identify tools.

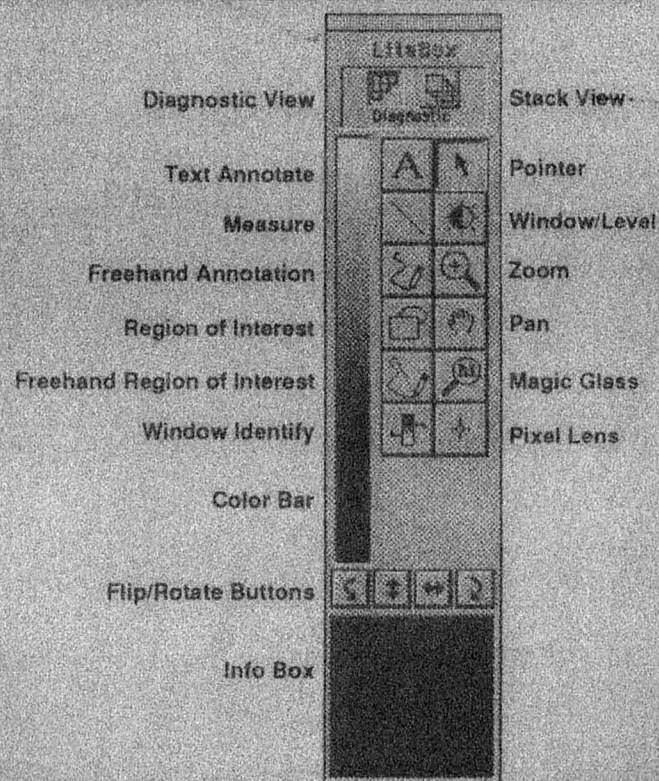

### KEY

### FUNCTION

- ⌘ A ..... Select All Images
- ⌘ B ..... Paste Info Box Text
- ⌘ D ..... Display Images
- ⌘ E ..... Deselect All
- ⌘ G ..... Diagnostic View
- ⌘ I ..... Show/Hide Info
- ⌘ K ..... Stack View
- ⌘ M ..... Window/Level Presets
- ⌘ O ..... Open Worklist
- ⌘ P ..... Print Report
- ⌘ Q ..... Quit the LiteBox Application
- ⌘ R ..... Rearrange
- ⌘ S ..... Save
- ⌘ W ..... Close Folder
- ⌘ Y ..... Show/Hide Annotation
- ⌘ Z or (F1) ..... Undo/Redo

### KEY

### FUNCTION

- ⌘ ] ..... Select Inclusive
- ⌘ = ..... Magnify Image 2X
- ⌘ - ..... Minify Image 2X
- ⌘ / ..... Reset Zoom to 1X
- ⌘ ; ..... Invert Gray Scale
- ⌘ 1 ..... Apply Window/Level Preset #1
- ⌘ 2 ..... Apply Window/Level Preset #2
- ⌘ 3 ..... Apply Window/Level Preset #3
- ⌘ 4 ..... Apply Window/Level Preset #4
- ⌘ 5 ..... Apply Window/Level Preset #5

### Report Processor

- F5 ..... Go Back
- F6 ..... Dictate Report
- F8 ..... Select Approvers
- F9 ..... Send
- F10 ..... Standard Reports
- F11 ..... Approve & Forward
- F12 ..... Approve & Complete

## IRIS KEYBOARD COMMAND REFERENCE

AOE		RP	
Command	Keyboard Equivalent	Command	Keyboard Equivalent
Quit .....	% Q	Quit .....	% Q
Undo .....	% Z or F1	Print Report .....	% P
Cut .....	% X or F2	Undo .....	% Z or F1
Copy .....	% C or F3	Cut .....	% X or F2
Paste .....	% V or F4	Copy .....	% C or F3
Select All .....	% A	Paste .....	% V or F4
Find Patient .....	return or enter	Select All .....	% A
Add Patient .....	% P	Go Back .....	F5
Update Patient .....	% U	Dictate Report .....	F6
Clear Form .....	esc	Select Approvers .....	F8
Add Exam .....	% E	Send .....	F9
Update Exam .....	% U	Standard Reports .....	F10
Print Barcode .....	% B	Approve & Forward .....	F11
<u>In the "Login" dialog:</u>		Approve & Complets .....	F12
"OK" Button .....	return or enter		
"Quit" Button .....	esc		

### RP

In the "Select Approvers" dialog:

Command	Keyboard Equivalent
"Add" Button .....	+
"Remove" Button .....	-
"OK" Button .....	return or enter
"Cancel" Button .....	esc
Select a radiologist name .....	1st few characters of name
Switch between lists .....	tab

In the "Standard Reports" dialog:

"Insert" Button .....	return or enter
"Cancel" Button .....	esc
Select a standard report name .....	1st few characters of name

